
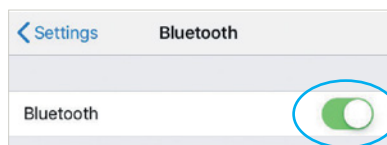


Connecting to a mobile device

Phone calls and media streaming

1 Bluetooth menu


Open the Bluetooth® menu on the phone (often found in “Settings”). Make sure Bluetooth is on.

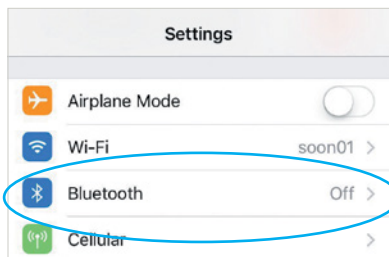

3 Phone calls

To accept a phone call, press the button on either device <2 seconds while the phone is ringing. If the hearing aids have tap control, double tap either ear to accept.

A long press (>2 seconds) on button, either ear will end or reject a call.

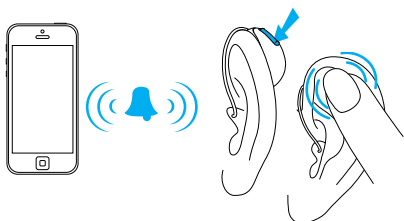

2 Pairing

Hearing aid name:

For hearing aids with a battery door, restart the hearing aid(s) by opening and closing the battery door.


For hearing aids without a battery door, turn the hearing aid(s) off by pressing the lower part of the button until the LED turns red (4 sec). Then turn on the hearing aid(s) by pressing the same button until the LED blinks green (2 sec).

The name of the hearing aid will appear on the phone. Tap to select, and click “pair” when prompted.



4 Adjusting volume

While using Bluetooth streaming, press the top button to increase the volume and the bottom button to decrease the volume. This can also be adjusted with the volume buttons on the side of the phone.

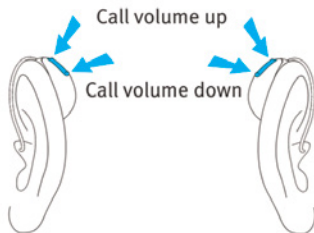

Unitron Remote Plus app

This app provides personalization and control of hearing aid settings using a smartphone. Adjust sound quality preferences with the comfort and clarity buttons and a 3-band equalizer, receive tips to assist with care and maintenance of your hearing aids, and more... for convenient, on-the-go personalization.

Get started in just a few taps


1 Download the app

Search for Unitron Remote Plus on the App Store or Google Play or scan the QR codes to the right.



2 Pair your hearing aids

Launch the app and follow the on-screen prompts.

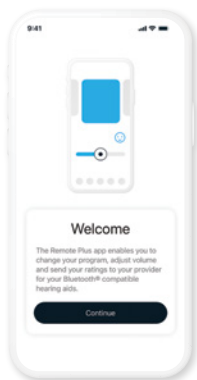

3 Unlock features

Stay engaged to maximize your hearing experience with Insights, a suite of unique features that enhance your hearing care experience for even greater personalization and support.



Enable notifications



Allow location access

Looking for more information? Visit unitron.com

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. Apple, the Apple logo, iPhone, and iOS are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google Inc.

028-6761-02/v4.00/2024-08/mk © 2024 Sonova AG, or its affiliates.

