

# Ultra/Super Power pre-set decision making guide

## Choosing the best pre-set with your client

### Instructions for client:

1. Tell us your listening preferences. Your hearing aids will be set to provide sound that is optimized for you and is the closest to how you want to hear right from the start.
2. Keeping in mind your most important listening situations, please answer the following questions.

### Questions:

#### When having a conversation in a quiet place, I prefer the speech to be:

Loud and full

☐
☐

Clear and distinct

#### When having a conversation in a noisy place, I prefer to hear:

A realistic balance of speech and noise

☐
☐

Clear speech and less background noise

#### When having a conversation in a noisy place, I prefer to:

Be fully aware of what's happening around me (including background noise)

☐
☐

Focus only on the conversation (and reduce the background noise)

#### When it comes to the volume of my hearing aids:

I often turn them up

☐
☐

I usually leave them as they are

#### When it comes to getting new hearing aids:

I want to keep hearing the same way I've always heard

☐
☐

I look forward to trying new hearing aid features

### Decision matrix:

4-5 answers in left column suggests using the **Classic** pre-set

2-3 answers in each column suggests using the **Conventional** pre-set

4-5 answers in right column suggests using the **Current** pre-set

Unitron makes life vibrant with amazing hearing solutions designed to make the experience easy. Because everyone deserves to **Love the experience.**<sup>™</sup>

**unitron.com**