












Unitron accessory portfolio (Effective November 2023)




Accessory		Compatible Devices	Description	Notes	Pairing
Moxi V-RS charger		All Moxi V-RS devices	Offering 24 hours of use on a single charge. The Moxi V-RS charger utilises magnetic contacts for easy insertion and removal. It's sleek and compact enough to fit into your pocket. Who is it best for? Those who prioritise design and style without having to forego the convenience of rechargeable technology.	All chargers come with a 3 year warranty. Utilises USB-C charging.	N/A
Moxi V charger		All Moxi V-R and Moxi V-RT devices	Offering 24 hours of use on a single charge for the confidence to go all day ¹ . The Moxi V charger is equipped with magnetic contacts for easy insertion and removal. Who is it best for? Those who need the easiest and most convenient solution to overcome management and dexterity issues.	All chargers come with a 3 year warranty. Utilises USB-C charging.	N/A
Charger Moxi R 3		All Blu, Discover Next and Discover Moxi rechargeable devices	Offering 24 hours of use on a single charge for the confidence to go all day ¹ . The Charger Moxi R 3 can house a drying capsule (available to order from Unitron) to manage moisture in the hearing instruments. Who is it best for? Perfect for those who have difficulty with batteries.	All chargers come with a 3 year warranty. Utilises Micro-USB charging.	N/A
Charger Combi BTE 2		Stride V-PR, Stride B-PR and DX Stride P R devices	Offering 24 hours of use on a single charge, the Charger Combi BTE 2 features an in-built voltage booster to ensure stable charging. Who is it best for? Perfect for those who have difficulty with batteries.	All chargers come with a 3 year warranty. Utilises Micro-USB charging.	N/A
Power pack		Charger Moxi R 3 and Charger Combi BTE 2	This accessory instantly transforms the Charger Moxi R 3 into a convenient, portable solution. The Power pack holds 7 full charges, without the need to plug the charging case into an outlet. Who is it best for? For those who require power on the go.	The power pack clips onto the bottom of the Charger Moxi R 3 and Charger Combi BTE 2.	N/A



¹ Actual battery life depends on several factors including hearing loss, battery age, and processing demands like direct connectivity.

Stride P R charger		All Vivante, Blu, Discover Next and Discover Stride rechargeable devices	<p>Offering 24 hours of use on a single charge for the confidence to go all day.¹ The Stride P R charger comfortably houses earmoulds.</p> <p>Who is it best for? Perfect for those who have large ear moulds.</p>	<p>All chargers come with a 3 year warranty.</p> <p>Utilises USB-C charging.</p>	N/A
TV Connector		All wireless Vivante, Blu, Discover Next Discover and T Moxi All devices.	<p>An easy plug and play solution that works with all TVs, allowing wireless streaming of the TV signal straight to the user's hearing instruments, all the while not interrupting the sound from the TV so users can enjoy watching their favourite shows and movies with others.</p> <p>Who is it best for? Those who want to watch their favourite shows and movies in high-quality stereo sound by wirelessly connecting their hearing instruments to the TV.</p>	<p>Multiple pairs of hearing instruments can connect to the same TV Connector simultaneously.</p> <p>Discover instruments can only be paired to one TV Connector at a time. Blu and Discover Next instruments can be paired to two TV Connectors.</p>	<p>Plug the TV Connector into an adequate power source (confirmed by a green light on the LED light indicator). Press the pairing button for 3 seconds and device/s will pair automatically.</p>
PartnerMic		All wireless Vivante, Blu, and Discover Next devices.	<p>The speaker's voice will be transmitted directly into the user's hearing instruments from up to 25 meters away. It uses its 3 microphones to create 2 beams that will focus towards the speech.</p> <p>Who is it best for? The PartnerMic is designed for use in environments where users want to be able to hear a single speaker in loud noise or at a distance.</p>	<p>The PartnerMic must be worn or held by the speaker at all times and positioned within 20cm of the speaker's mouth for optimal performance.</p>	<p>Ensure the PartnerMic is fully charged and switch it on. Press the pairing button for 3 seconds (confirmed by a flashing blue light on the LED light indicator), and device/s will pair automatically.</p>

Remote Control		All wireless Vivante, Blu, and Discover Next devices.	<p>Gives users the ability to easily and discreetly make simple adjustments to volume or programs.</p> <p>Who is it best for? For those who want an easy-to-use remote control or those who cannot or do not want to use the Remote Plus app for adjustments.</p>	<p>The battery is a replaceable cell, not rechargeable.</p> <p>The Remote Control requires one lithium button cell (CR2032).</p>	Hold down the + button and turn the Remote Control on at the same time, this will put the Remote Control into pairing mode (confirmed by a flashing blue light on the LED light indicator), device/s will pair automatically.
IIC remote		Non-wireless Blu and Discover Next Inera 10A Omni devices	<p>The pocket-sized IIC remote gives you a choice between magnetically adjusting between programs, volume up only or volume down only (programmable in Truefit).</p> <p>Who is it best for? For those who like the added convenience of having further control over their hearing.</p>	IIC Remote function must be requested at the time of hearing instrument order.	N/A
Remote Plus App		All wireless Vivante, Blu, Discover Next and Discover devices.	<p>Users can access the Remote Plus app to make adjustments to essential features and share their hearing experiences with their clinician in real time, it also enhances their hearing experience with on-demand tips, troubleshooting and videos on how to care for and use their hearing instruments.</p> <p>Who is it best for? Those who want to control their hearing instruments and share in-the-moment impressions of their hearing instrument's performance, all from their smartphone.</p>	<p>Requires iOS 12 or newer or Android version 7 or newer, with Bluetooth 5.0 or newer.</p> <p>Remote Plus app functionality will differ depending on what device is paired.</p>	<p>Please click on the links to access Remote Plus pairing guides</p> <p>Android pairing iOS pairing</p>

Tempus accessory portfolio: please note that all Tempus accessories have been discontinued as of Dec 2022

Accessory		Compatible Devices	Description	Notes	Pairing
uDirect 3		All wireless Tempus and North devices (excl. T Moxi All)	Provides connectivity with superior battery life to mobile phones, tablets, TVs, FM receivers and MP3 players while conveniently acting as a remote. It is worn around the neck on a neck loop and offers 24 hours of streaming to the wearer. Who is it best for? Those who want to control their hearing instruments and enjoy long periods of stereo streaming from their entertainment devices, like the TV.		uDirect 3 is paired to the Hearing Instruments via Truefit software in the Instruments -> Accessories tab. A mini-USB cable (available to order from Unitron) is required.
uTV 3			Wirelessly sends stereo audio from the television directly to both hearing instruments through uDirect 3 for easy, enjoyable listening. The uTV 3 has a working range of up to 30 meters and can be used to control the volume of the streamed audio signal. Who is it best for? For those who want to enjoy high definition stereo streaming from the TV.	Requires a uDirect 3 or uStream in order to communicate with the hearing instruments. uTV 3 acts as a cradle to conveniently charge and store the uDirect 3 when not in use.	Pair uDirect 3 or uStream to the hearing instruments in the Truefit software first. Save a new session and close down Truefit. Plug the uTV 3 into an adequate power source (confirmed by a green light on the LED light indicator). Press the pairing button for 3 seconds on both the uTV 3 (confirmed with a flashing Blue light on the LED light indicator) and uDirect3 or uStream (confirmed by flashing lights on the front of the device) and accessories will pair automatically.
Remote Control 2			Perfect for those who want discreet, straightforward control over the essential features such as programs and volume on their hearing instruments. Who is it best for? For those who want an easy-to-use remote control.	The battery is a replaceable cell, not rechargeable. The Remote Control 2 requires one AAA battery.	Remote Control 2 is paired to the Hearing Instruments via Truefit software in the Instruments -> Accessories tab. A mini-USB cable (available to order from Unitron) is required.

uStream			<p>An alternative for the uDirect3, this small clip-on accessory provides wireless streaming to mobile phones, tablets, TVs, FM receivers and MP3 players. uStream can also be used as a remote control.</p> <p>Who is it best for? Those who do not want to wear a neckloop and prefer discreet control of their hearing instruments and stereo streaming. Best for those who do not wish to stream for long periods of time.</p>	<p>Optimum working range for the uStream is within 40cm of the hearing aids. It offers 4 hours of streaming for the wearer.</p>	<p>uStream is paired to the Hearing Instruments via Truefit software in the Instruments -> Accessories tab. A mini-USB cable (available to order from Unitron) is required.</p>
uMic 2		All wireless Tempus and North devices (excl. T Moxi All)	<p>A personal wireless microphone system that lets you hear your companion easily in challenging listening environments. The uMic is attached to the speaker using a built-in clip and it wirelessly sends audio directly to both hearing instruments. The uMic2 has an omni-directional microphone and has a range of up to 20 meters.</p> <p>Who is it best for? For those who want direct audio from a single sound source to their hearing instruments in any challenging environments or to help overcome the challenges of distance between the speaker and hearing aid wearer.</p>	<p>Requires a uDirect 3 or uStream in order to communicate with the hearing instruments.</p> <p>uMic 2 works best when worn or held by the speaker.</p>	<p>Pair uDirect 3 or uStream to the hearing instruments in the Truefit software first. Save a new session and close down Truefit. Put the uDirect 3 or uStream into pairing mode by pressing the pairing button for 3 seconds (confirmed with a flashing Blue light on the LED light indicator).</p> <p>Activate pairing mode on the uMic by holding down the + button and switching on the uMic at the same time (confirmed by flashing blue light) and accessories will pair automatically.</p>