

Unitron TrueFit™ software advanced installation guide

Command Line Parameters

Table of contents

- 1. Introduction2
- 2. Installer prerequisites2
- 3. Unitron TrueFit installer command line parameters3
 - 3.1 Available properties3
 - 3.2 Examples5
- 4. Setup.exe command-line parameters6
- 5. Msiexec.exe command-line parameters6
- Need assistance?7

1. Introduction

The Unitron TrueFit software installer (InstallShield project type: Basic MSI) consists of an MSI and a setup.exe. The setup.exe is responsible for checking and installing prerequisite components which can be found in the 'ISSetupPrerequisites' directory of the installer DVD. Both, the MSI and setup.exe, react to command line parameters. Therefore, when launching the installer via the setup.exe, one has to respect a special syntax in order to have the arguments for the MSI passed through to it (-> chapter 4, /v : pass arguments to Msiexec).

The setup.exe command line parameters are described in chapter 4. The MSI command line options are described in chapter 5. In addition, Unitron TrueFit package-specific properties can be specified in order to configure the installation itself. These properties are described in chapter 3, together with some examples.

All this information is subject to change without further notice.

2. Installer prerequisites

When installing Unitron TrueFit software via the standard DVD package MSI installer with setup.exe, the installer prerequisites are handled automatically by the installation program.

If you are using a single file MSI installer, the prerequisites are not automatically installed and it is expected that all prerequisites are properly deployed prior to attempting to install Unitron TrueFit software.

The following table lists all of Unitron TrueFit software's prerequisites, and the recommended order in which they should be installed.

- Windows Installer version 3.1.4
- Microsoft .NET Framework 4.8 (Full)
- Microsoft Visual C++ 2012 Update 4 Redistributable
- Microsoft Visual C++ 2017 Redistributable

3. Unitron TrueFit installer command line parameters

3.1 Available properties

The Unitron TrueFit MSI takes the following properties into account:

CHECKSOFF < > 1: default, checks turned on

1: the following checks are turned off

- required operating system
- admin rights
- prerequisites (windows installer, .net framework, runtime libs...)
- installer was launched from setup.exe (if applicable)

C_COUNTRY Unitron TrueFit country selection. Not all listed countries might be supported at all times. Please verify the actual availability in the 'User Settings' dialog, country dropdown menu.

Algeria	DZ	Fiji	FJ	Luxembourg	LU	Saudi Arabia	SA
Argentina	AR	Finland	FI	Macau	MO	Serbia	RS
Australia	AU	France	FR	Macedonia	MK	Singapore	SG
Austria	AT	Germany	DE	Malaysia	MY	Slovak Republic	SK
Bahrain	BH	Greece	GR	Malta	MT	Slovenia	SI
Bangladesh	BD	Hong Kong	HK	Mexico	MX	South Africa	ZA
Belgium	BE	Hungary	HU	Moldova	MD	Spain	ES
Bolivia	BO	Iceland	IS	Mongolia	MN	Sri Lanka	LK
Bosnia	BA	India	IN	Morocco	MA	Sudan	SD
Brazil	BR	Indonesia	ID	Myanmar	MM	Sweden	SE
Bulgaria	BG	Iran	IR	Namibia	NA	Switzerland	CH
Canada	CA	Iraq	IQ	Netherlands	NL	Syria	SY
Chile	CL	Ireland	IE	New Zealand	NZ	Taiwan	TW
China	CN	Israel	IL	Norway	NO	Thailand	TH
Colombia	CO	Italy	IT	Oman	OM	Tunisia	TN
Costa Rica	CR	Japan	JP	Pakistan	PK	Turkey	TR
Croatia	HR	Jordan	JO	Palestine	PS	Ukraine	UA
Cyprus	CY	Kazakhstan	KZ	Panama	PA	United Kingdom	GB
Czech Republic	CZ	Kenya	KE	Peru	PE	Uruguay	UY
Denmark	DK	Korea	KR	Philippines	PH	Uzbekistan	UZ
Dominican Republic	DO	Kuwait	KW	Poland	PL	United Arab Emirates	AE
Ecuador	EC	Latvia	LV	Portugal	PT	United States	US
Egypt	EG	Lebanon	LB	Qatar	QA	Venezuela	VE
El Salvador	SV	Libya	LY	Romania	RO	Vietnam	VN
Estonia	EE	Lithuania	LT	Russia	RU	Yemen	YE

C_LANGUAGE Unitron TrueFit language selection. Not all listed languages might be supported at all times. Please verify the actual availability in the ‘User Settings’ dialog, language dropdown menu.

Arabic	ar-JO	Japanese	ja-JP
Bulgarian	bg-BG	Korean	ko-KR
Chinese (Simplified)	zh-CN	Norwegian	nb-NO
Chinese (Traditional)	zh-TW	Polish	pl-PL
Czech	cs-CZ	Portuguese	pt-PT
Danish	da-DK	Portuguese (Brazil)	pt-BR
Dutch	nl-NL	Romanian	ro-RO
English	en-US	Russian	ru-RU
Farsi	fa-IR	Serbian S	sr-Latn-C
Finnish	fi-FI	Slovakian	sk-SK
French	fr-FR	Spanish	es-ES
French (Canada)	fr-CA	Spanish (Latin America)	es-CO
German	de-DE	Swedish	sv-SE
Greek	el-GR	Turkish	tr-TR
Hungarian	hu-HU	Ukrainian	uk-UA
Italian	it-IT		

C_NOAHINSTALLATION

0: standalone installation

1: Noah installation (default setting but not considered if Noah not found)

C_INSTALLDIR Unitron TrueFit installation directory

C_FEATUREKEYS OfflineMode: Sets TrueFit to Offline Mode. Prevents Unitron TrueFit from accessing the Internet.

DisableImprovementProgram: Disables data collection for the Unitron TrueFit improvement program.

Note: Multiple C_FEATUREKEYS can be used within a single command by separating them with a semi-colon.

3.2 Examples

Precondition: Current directory is where the installation program resides.

Example 1:

The following command will install Unitron TrueFit...

- unattended/silent
- will set Unitron TrueFit country to France
- will set Unitron TrueFit language to French
- will register Unitron TrueFit in Noah (Noah needs to be installed)
- will use the default install directory
- will produce a install log file UnitronInstall.log in the Temp directory
- will set Unitron TrueFit to Offline Mode
- will disable data collection for the Unitron TrueFit improvement program

```
setup.exe /s /v"/qn /l*v %temp%\UnitronInstall.log C_COUNTRY=FR C_LANGUAGE=fr-FR  
C_NOAHINSTALLATION=1 C_FEATUREKEYS="OfflineMode;DisableImprovementProgram"
```

Or, for a single file MSI installer:

```
msiexec /i "Unitron TrueFit.msi" /qn /l*v %temp%\UnitronInstall.log C_COUNTRY="FR"  
C_LANGUAGE="fr-FR" C_NOAHINSTALLATION="1"  
C_FEATUREKEYS="OfflineMode;DisableImprovementProgram"
```

Example 2:

The following command will install Unitron TrueFit software...

- unattended, with a basic user interface
- will set Unitron TrueFit country to Germany
- will set Unitron TrueFit language to German
- will install Unitron TrueFit in Standalone with desktop shortcut
- will set the installation directory to C:\Program Files\Unitron\Unitron TrueFit
- will produce a install log file UnitronInstall.log in the Temp directory

```
setup.exe /s /v"/qb /l*v %temp%\UnitronInstall.log C_COUNTRY=DE C_LANGUAGE=de-DE  
C_NOAHINSTALLATION=0
```

```
C_INSTALLDIR="C:\Program Files (x86)\Unitron\Unitron TrueFit\""
```

Example 3:

The following command will uninstall Unitron TrueFit with a basic user interface and produce a log file C:\UnitronUninstall.log

```
setup.exe /s /x /v"/qr /l*v C:\UnitronUninstall.log"
```

Note: when the setup.exe is called and no log file defined, a log file %temp%\UnitronInstall.log is automatically created.

4. Setup.exe command-line parameters

https://docs.revenera.com/installshield24helplib/helplibrary/IHelpSetup_EXECmdLine.htm

5. Msiexec.exe command-line parameters

<https://docs.microsoft.com/en-us/windows/win32/msi/command-line-options>

-  Instructions are available at www.unitron.com/instr in Adobe® Acrobat® PDF format.
To view them, you must have Adobe Acrobat Reader installed. Visit Adobe.com to download.
-  To obtain a free paper copy of the instructions, please contact Unitron using the phone numbers or address in this guide. A copy will be sent within 7 days.

Need assistance?

Australia - 1800 212 313
Austria - +49 711 658538 0
Belgium - +31 88 600 88 10
Brazil - +55 11 3748 0960
Canada - 1 800 265 8255
China - +86 21 61205533
Denmark - +45 64 41 78 87
EU Representative - +49 711 658538 0
France - 04 26 23 22 00
Germany - +49 711 658538 0
India - +91-22-26871151

For countries not listed - +1 519 895 0100

Netherlands - +31 88 600 88 10
New Zealand - 0800 864 8766
Norway - +47 22 47 76 30
Russia - +7 495 788 02 04
South Africa - (011) 467 7662/52
South Korea - +82 2 859 4902
Spain - +34 902 494 960
Sweden - +46 (0) 8 546 20 960
Switzerland - 0800 928 800
U.K. - +44 (0) 1925 247810
U.S. - 1 800 888 8882

Contact your Unitron representative for more detailed information related to Unitron TrueFit software v5.3 updates.



Sonova AG
Laubisrütistrasse 28
CH-8712 Stäfa, Switzerland



Importer for European Union:
Sonova Deutschland GmbH
Max-Eyth-Str. 20
70736 Fellbach-Oeffingen, Germany

sonova
HEAR THE WORLD