

Remote Plus app

**You choose how
you want to
hear your world**



A Sonova brand

Embrace your hearing journey for what it truly is: **it's your experience**

And when you do, you'll discover that the Remote Plus app makes it easy for you to customise your hearing journey your way, every step of the way.

*For the use of the Remote Plus app, Unitron hearing aids with Bluetooth® connectivity are required. The Remote Plus app works on Apple smartphones with iOS 15 or newer and Android smartphones with version 8.0 or newer.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of each marks by Sonova AG is under license.


Get started in just a few taps

1



First, search for Unitron Remote Plus on the App Store or Google Play or scan the QR codes above, and download the app to your smartphone.

2

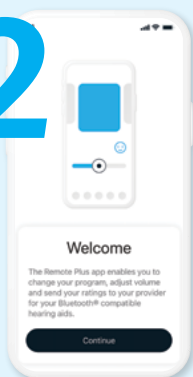

Launch the app and follow the on-screen prompts to pair your hearing aids.

3




**Enable
notifications**



**Allow
location access**

Stay engaged to maximise your hearing experience with Insights, a suite of unique features that enhance your hearing care experience for even greater personalisation and support!

App Store badge is a trademark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC.

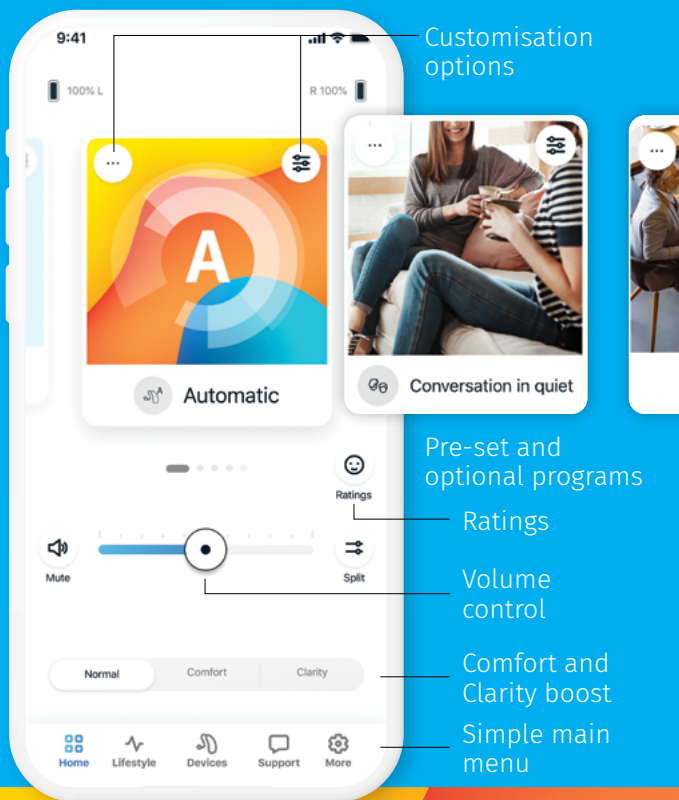


Personalise your experience in any moment

The Remote Plus app allows you to adjust sound **Clarity** and listening **Comfort** in the moment. From the home screen, you can swipe to select pre-set and optional programs designed for different locations, times of day, conversations or lifestyle activities.

Tap **Customisation options** to access sound and equaliser controls for further optimisation of your listening experience.



Restaurant

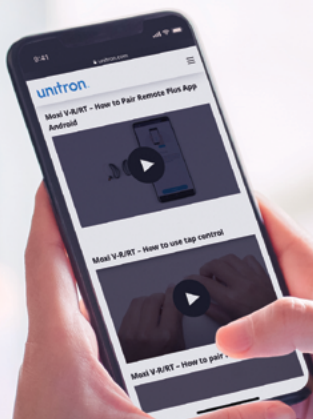

Take confident steps into better hearing

It might feel like there is a lot to learn at your first appointment, but don't worry about remembering it all. As you begin your hearing journey, **Coach**, the app's virtual guide, will deliver easy-to-understand tips, reminders and advice.

And as you continue to wear your hearing aids, have the peace of mind that if you misplace or lose them, **Find my Devices** can help you locate them.



Work together throughout your hearing journey

Stay in touch with your hearing care professional with **Ratings**. You can easily provide in-the-moment feedback about your experience. Your hearing care provider can access insightful listening lifestyle data via **Success Check**, and they can even send fine-tuning adjustments directly to your app, wherever you are, via **Remote Adjust**.



With quick and seamless navigation in the Remote Plus app, you can easily and discretely make the adjustments you need in the moment. From volume control to programs that you can select and customise, you choose how to personalise your experience!

You can even customise the app's user interface by choosing between two modes to suit your desired level of functionality.

unitron TM **Love the
experience**

Unitron makes life vibrant with amazing hearing solutions designed to make the experience easy. Because everyone deserves to **Love the experience.** TM

© 2025 Sonova AG, or its affiliates.

028-6502-48/V5.00/2025-02/jb

unitron.com/au
1800 212 313



Sonova AG,
Lindberghstrasse
28, CH-8112
Stäfa, Switzerland



sonova
HEAR THE WORLD