

Unitron Remote Plus app

With quick and seamless navigation in the Remote Plus app, you can easily and discretely make the adjustments you need in the moment. From volume control to programs that you can select and customise, you choose how to personalise your experience!

Get started in just a few taps


1 Download the app

Search for Unitron Remote Plus on the App Store or Google Play or scan the QR codes to the right.



2 Pair your hearing aids

Launch the app and follow the on-screen prompts. You may need to follow additional steps on the next page to complete the pairing process.

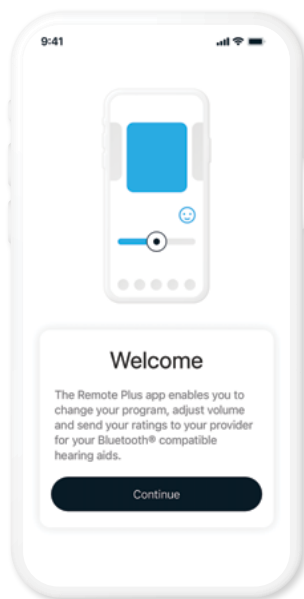

3 Enable notifications

Allow location access

Stay engaged to maximise your hearing experience with innovative features like Coach, your virtual hearing aid guide, while Find my Devices gives you peace of mind by helping you keep track of misplaced or lost hearing aids.


Looking for more information?
Visit unitron.com

Connecting to a mobile device

Phone calls and media streaming

1 Bluetooth menu


Open the Bluetooth® menu on the phone (often found in “Settings”).
Make sure Bluetooth is on.

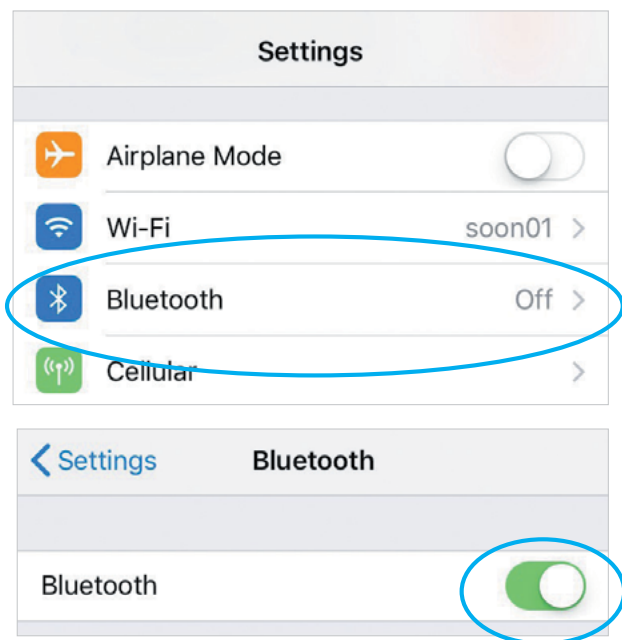

3 Phone calls

A short press on the push button will accept an incoming call and a long press will reject an incoming call.

If the hearing aids have tap control, double tap either ear to accept a call. A call can be ended by double tapping your ear again.

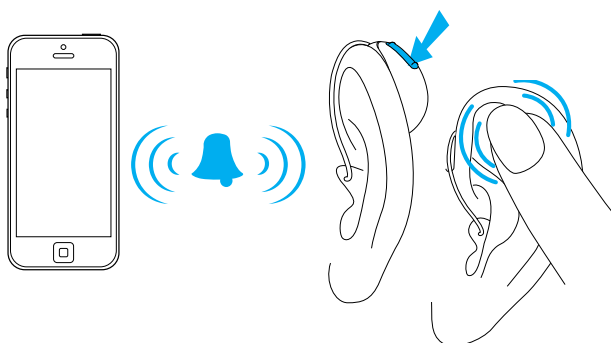

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc.

Apple, the Apple logo, iPhone, and iOS are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Android, Google Play, and the Google Play logo are trademarks of Google Inc.

028-6761-48/v5.00/2025-12/jb © 2024–2026 Sonova AG, or its affiliates.


2 Pairing

Hearing aid name:

For rechargeable hearing aids, turn off each device by holding the button or lower part of the button until the LED turns red (up to 4 sec.). Then turn on each hearing aid by holding the button or lower part of the button for up to 4 seconds until the solid green light indicates the hearing aid is ready.

For hearing aids with a battery door, enable pairing mode by opening and closing the battery door for each hearing aid.

The name of the hearing aid will appear on the phone. Tap it to pair.

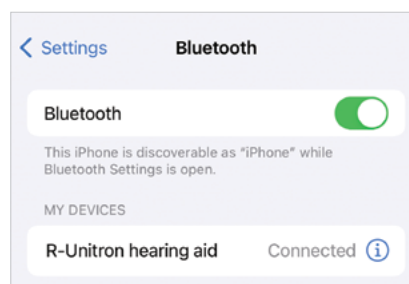

4 Adjusting volume

While using Bluetooth streaming, press the button or upper and lower parts of the button to adjust the volume. This can also be adjusted with the Volume Control in the app, or with the volume buttons on the side of the phone.

