Unitron TrueFit™ fitting software

Successful fittings. Simplified.

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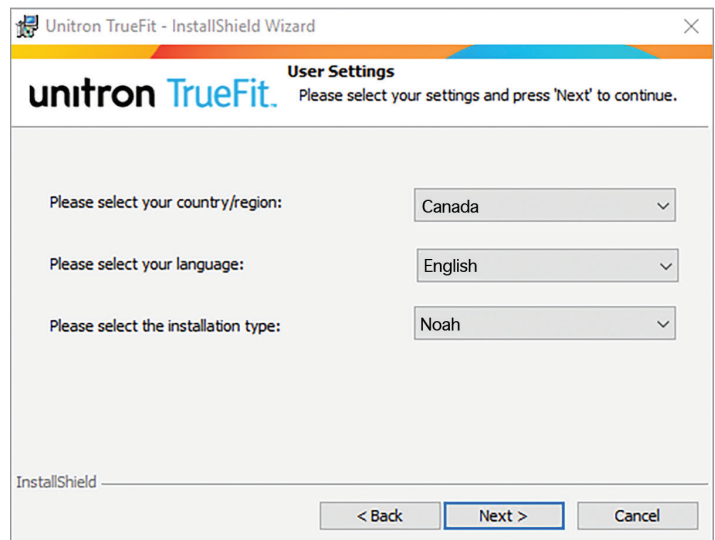
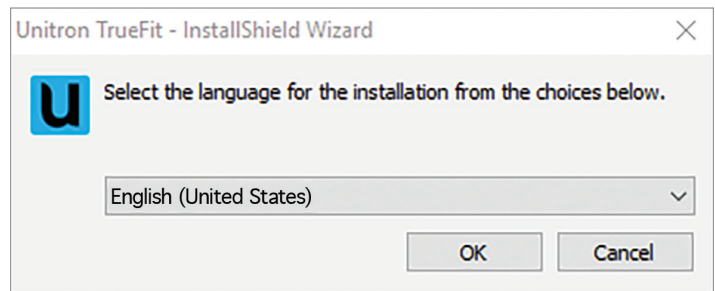
Installing Unitron TrueFit Software

1. Insert the Unitron TrueFit disk/USB or download and extract the installation file.
 - If the installation program does not start automatically, double click on setup.exe from the fitting software disk/USB or in the downloaded folder.
 - Follow the displayed step by step instructions. InstallShield installs the fitting software on your PC.
2. Select the preferred installation language: English, French, German or Dutch (for the installation procedure only) and click **OK**.
3. If additional Microsoft software components are required, they will be listed. Click **Install** to install them. If you are prompted to reboot, please follow the step-by-step instructions.
4. When the InstallShield Wizard appears, click **Next >**.
5. Read the license agreement and accept it to continue the installation process. Click **Next >**.
6. When the User Settings dialog appears, follow the instructions on the screen to select your country, your language and the installation type. Click **Next >**.
 - If NOAH 4 is installed, you can choose to install Unitron TrueFit under NOAH or Standalone, otherwise Standalone will be pre-selected.
7. Choose **Next >** on the Custom Setup screen to install all features in the default location, or follow the instruction to make changes.
8. Choose **Install** to begin the installation.
9. When prompted, select **Finish** to complete the installation.

System Requirements

Processor	Intel Core or higher performance
RAM	8 GB or more
Hard disk space	4 GB or more
Operating systems supported	› Windows 11, Home / Pro / Enterprise / Education › Windows 10, Home / Pro / Enterprise / Education
Display resolution	1280 x 1024 pixels
Patient secondary display resolution	1366 x 768 pixels
Graphic card	16 Million (24 bit) screen colors or more
Drive for manual software installation	› DVD › USB
Serial COM port	One for HI-PRO if used via serial COM port
USB ports	One for each purpose: › Bluetooth adaptor › Accessory programming › HI-PRO USB / HI-PRO 2 / iCube II / Noahlink Wireless
Programming interfaces	› iCube II › NOAHlink / Noahlink Wireless › HI-PRO / HI-PRO USB / HI-PRO 2
NOAHlink driver	Latest version available
NOAHlink Wireless driver	Latest version available
Internet connection	Highly recommended
Sound card	Stereo
Playback system	20 Hz – 14 kHz
Noah version	Latest version (Noah 4.4 or higher) Please check the Noah limitations for Windows operating systems on http://www.himsa.com
Automatic REM	Noah Version 4.4.2280 or higher Natus Otosuite 4.81.00 or higher Natus Aurical FreeFit for REM & Aurical HIT for test box measurements

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Sonova AG
Sonova Deutschland GmbH