


Remote Plus app

You choose how
you want to hear
your world

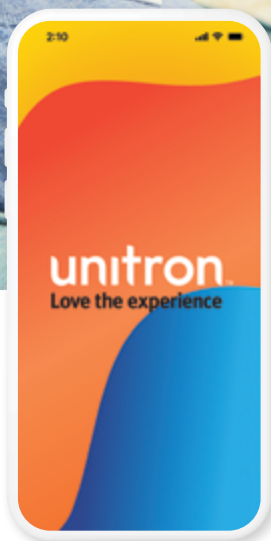

A Sonova brand

Embrace your hearing journey for what it truly is: **it's your experience**

And when you do, you'll discover that the Remote Plus app makes it easy for you to customise your hearing journey your way, every step of the way.

*For the use of the Remote Plus app, Unitron hearing aids with Bluetooth® connectivity are required. The Remote Plus app works on Apple smartphones with iOS 15 or newer and Android smartphones with version 8.0 or newer.



With quick and seamless navigation in the Remote Plus app, you can easily and discretely make the adjustments you need in the moment. From volume control to programs that you can select and customise, you choose how to personalise your experience!

You can even customise the app's user interface by choosing between two modes to suit your desired level of functionality.


Get started in just a few taps

1



First, search for Unitron Remote Plus on the App Store or Google Play or scan the QR codes above, and download the app to your smartphone.

2

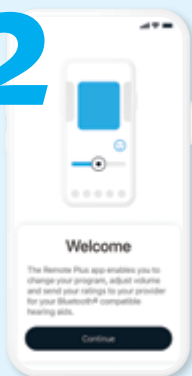

Launch the app and follow the on-screen prompts to pair your hearing aids.

3



**Activate
Insights**




**Enable
notifications**



**Allow location
access**

These three options unlock a suite of unique features that enhance both your app and your hearing care experience for even greater personalisation and support!

App Store badge is a trademark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC.

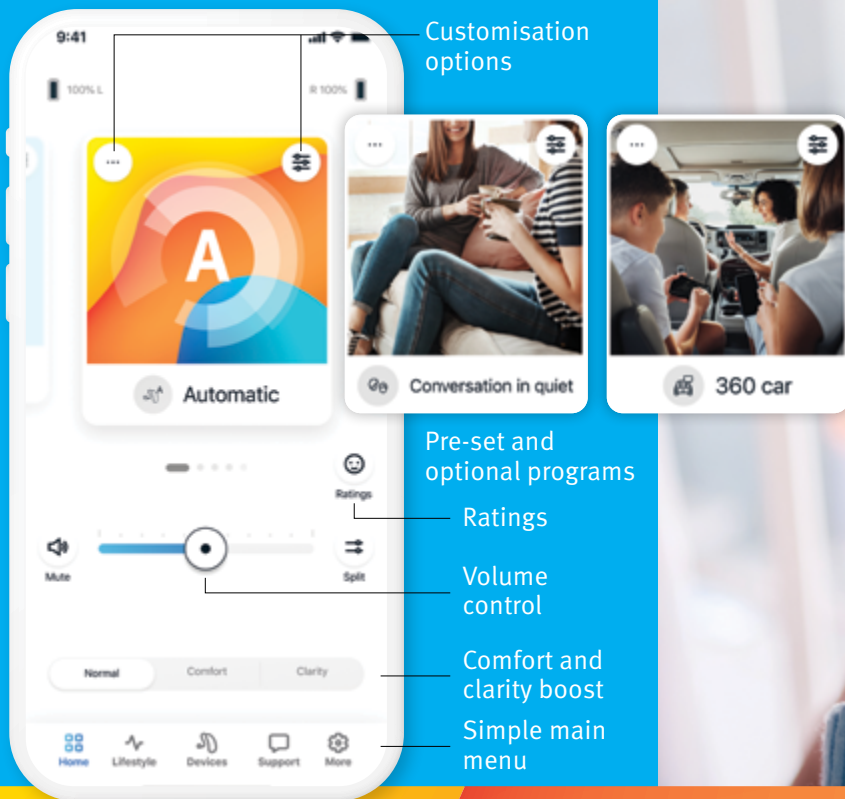

Personalise your experience in any moment

Take confident steps into better hearing

Work together throughout your hearing journey

The Remote Plus app allows you to adjust sound **clarity** and listening **comfort** in the moment. From the home screen, you can swipe to select pre-set and optional programs designed for different locations, times of day, conversations or lifestyle activities.

Tap **customisation options** to personalise the look and feel of your program carousel, and to access sound and equaliser controls for further optimisation of your listening experience.

It might feel like there is a lot to learn at your first appointment, but don't worry about remembering it all. As you begin your hearing journey, you can confidently manage the day-to-day maintenance of your hearing aids with the help of the app-based **Coach** notifications. You'll have access to easy-to-understand instructions and videos, plus you'll receive reminders and tips delivered straight to your smartphone.

Stay in touch with your hearing care professional with **Ratings**. You can easily provide real-time feedback about your experience. Your hearing care provider can access insightful listening lifestyle data via **Success Check**, and they can even send fine-tuning adjustments directly to your app, wherever you are via **Remote Adjust**.



At Unitron, we empower people with life-enhancing hearing experiences that fit seamlessly into their world. Our sound performance technology, Experience Innovations, and intuitive design work perfectly together to provide personalisation and optimisation. Because everyone deserves to **Love the experience.**TM

© 2024 Sonova AG, or its affiliates.

028-6502-02/V3.00/2024-04

1800 212 313
unitron.com/au



Sonova AG,
Lauderstrasse
26, CH-8712
Olten, Switzerland



sonova
HEAR THE WORLD