Unitron TrueFit™ fitting software

Successful fittings. Simplified.

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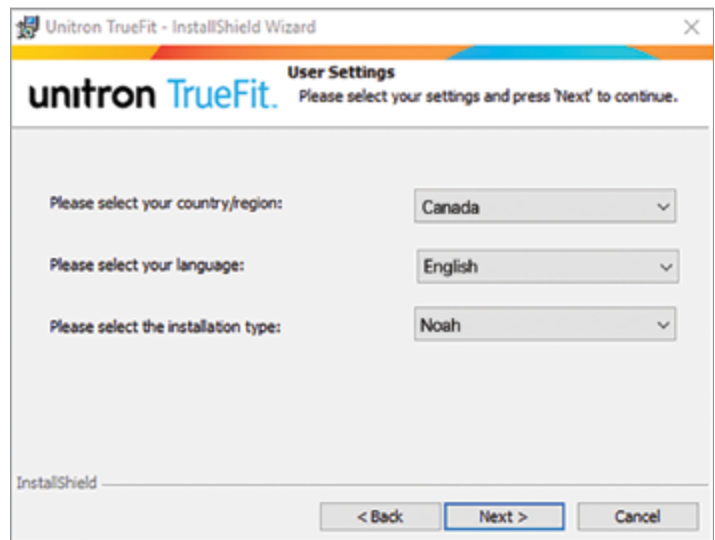
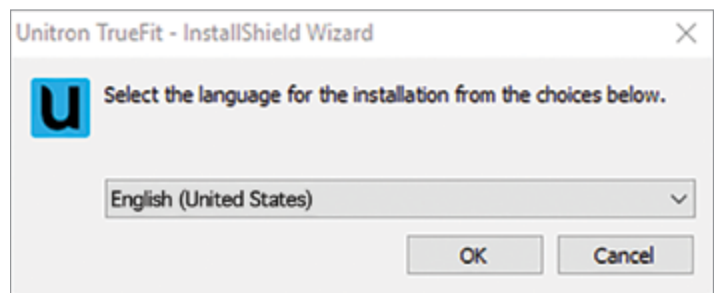
Installing Unitron TrueFit Software

1. Insert the Unitron TrueFit USB or download and extract the installation file.
 - If the installation program does not start automatically, double click on setup.exe from the Unitron fitting software USB or in the downloaded folder.
 - Follow the displayed step by step instructions. InstallShield installs the fitting software on your PC.
2. Select the preferred installation language: English, French, German or Dutch (for the installation procedure only) and click **OK**.
3. If additional Microsoft software components are required, they will be listed. Click **Install** to install them. If you are prompted to reboot, please follow the step-by-step instructions.
4. When the InstallShield Wizard appears, click **Next >**.
5. Read the license agreement and accept it to continue the installation process. Click **Next >**.
6. When the User Settings dialog appears, follow the instructions on the screen to select your country, your language and the installation type. Click **Next >**.
 - If Noah 4™ is installed, you can choose to install Unitron TrueFit under Noah or Standalone, otherwise Standalone will be pre-selected.
7. Choose **Next >** on the Custom Setup screen to install all features in the default location, or follow the instruction to make changes.
8. Choose **Install** to begin the installation.
9. When prompted, select **Finish** to complete the installation.

System Requirements

Operating systems supported	> Windows 11, Home / Pro / Enterprise / Education > Windows 10, Home / Pro / Enterprise / Education
Processor	Intel Core or higher performance
RAM	8 GB or more
Hard disk space	4 GB or more
Display resolution	1280 x 1024 pixels
Patient secondary display resolution	1366 x 768 pixels
Graphic card	16 Million (24 bit) screen colors or more
Drive for manual software installation	> USB
Serial COM port	One for HI-PRO if used via serial COM port
USB ports	One for each purpose: > Bluetooth adaptor > Accessory programming > HI-PRO USB / HI-PRO 2 / iCube II / Noahlink Wireless / Noahlink Wireless 2
Programming interfaces	> iCube II > NOAHlink / Noahlink Wireless / Noahlink Wireless 2 > HI-PRO / HI-PRO USB / HI-PRO 2
NOAHlink driver	Latest version available
Noahlink Wireless driver	Latest version available
Internet connection	Highly recommended
Sound card	Stereo
Playback system	20 Hz – 14 kHz
Noah version	Latest version (Noah 4.4 or higher) Please check the Noah limitations for Windows operating systems on http://www.himsa.com
Automatic REM	Noah version 4.4.2280 or higher Auditdata Measure version 6.0 or higher Auditdata Primus version 4.1 or higher Interacoustics Affinity Suite version 2.19 or higher (excludes 2.20) Interacoustics Callisto Suite version 1.20 or higher MedRX Studio version 1.2.1 or higher Natus® Otosuite version 4.82 and higher Signia Unity 3 version 5.9 or higher Signia Unity 4 version 6.0 or higher

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Sonova AG
Sonova Deutschland GmbH